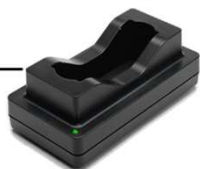
REF 60501

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INSTRUCTIONS FOR USE - ENGLISH

This *Instructions For Use manual* is the most comprehensive source of information for the safe and effective use of your product. Keep and consult this reference manual during the life of the product.

IMPORTANT INFORMATION

The words WARNING, CAUTION, and NOTE have special meaning and should be reviewed.

WARNING	A WARNING highlights a safety-related issue. ALWAYS comply with this information to prevent patient and/or healthcare staff injury.
CAUTION	A CAUTION highlights a product reliability issue. ALWAYS comply with this information to prevent product damage.
NOTE	A NOTE supplements and/or clarifies procedural information.

COMPLIANCE STATEMENT

Medical Device acc. Directive 93/42/EEC

This device is subject to the conformity assessment procedure according to annex VII of Directive 93/42/EEC.

INDICATION FOR USE

This smart charger is a component of the **Vivi®** System, to be used by healthcare professionals outside of the patient environment to charge the Battery Pack HW REF 60402.

CONTRAINDICATIONS

None known.

FOR USE WITH

Use this smart charger to charge the Battery Pack HW REF 60402.

USER/ PATIENT SAFETY

WARNINGS:

- This battery charger may not be located in the vicinity of a patient. In accordance with requirements defined in EN60601-1-1, the horizontal distance to a patient must be at least 1.5 m, the vertical at least 2.5 m.
- Use the battery charger only in dry indoor environments! Wet and moisture covered products should not be used!
- FCC Advice, valid for USA:* This product has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation.
- DO NOT sterilize.
- MAKE SURE that the power supply is easily accessible when connected to the mains socket.

SPECIFICATION	Power Supply	Single-Bay Smart Charger
REF (Type)	30601	60501
Input Voltage	100-240VAC/ 50-60 Hz	19-26VDC / 3.4 A max
Charging and Output Voltages	19VDC $\pm$ 5% / 3.4A	0 – 17.4 VDC 0 – 4.8 A
Power	65W	50W
Dimensions (LxWxH)	95 x 51 x 25.4 mm	151 x 90 x 43 mm
Operating / Storage Temperature	0°C to 40°C / -10°C to 70°C	0°C to 40°C / -10°C to 70°C
Weight	270g	198g
Safety/EMC Standards	EN60950-1 / UL60950-1* / EN55011 / EN55022 / EN55024* / FCC15 Class B / EN61000-4-2,-3,-4,-5,-6,-8,- 11* / EN61000-3-2,3-3*, EN60601-1 / EN60601-1-2	

*...valid only for power supply

SYMBOL DEFINITIONS

The symbols located on the equipment and/or labelling are defined in this section.

SYMBOL	DEFINITION
	Caution, consult accompanying documents
	Consult instructions for use
	Mark of conformity to applicable European Directives
	Manufacturer
	Batch number
	Quantity
	Article number
	Non sterile
	Use only in enclosed spaces (indoors)
	Dispose acc. to WEEE directive (EU) or acc. local guidelines
	DC Voltage
	China RoHS
	Compliance to California Energy Commission requirements

HOW TO USE

MAKING CONNECTIONS

- Connect the cable from the power supply to the battery charger.
- Connect the power cable to the power supply and plug it into the mains socket.
- The charger does perform as self test. After completion, the LED is going out. The battery charger is ready for use.
- If a red LED remains on and no battery is in the charging bay, the battery charger is defective.

CHARGING

- Connect the battery charger as described above.
- Place a discharged Battery Pack into the battery charger.
- The battery charger runs a battery recognition and initialization cycle. LED's might go on and off during this cycle.
- The battery will be charged. Leave the battery until the solid green LED appears. **NOTE:** We recommend leaving the battery pack in the charging bay if it is intended for use.

CHARGING INDICATION (LED)

One time Red/ Orange/ Green	Self test: Charger is ready for use (after LED goes out)
Red/ Green blinking	The battery is not recognized as Smart Battery. Either a conventional battery has been inserted or an extremely discharged Smart Battery. If it is a Smart Battery, it will be reactivated within 15 minutes and recharged. If it is not the case, the LED with turn to solid red – see below.
Orange blinking	The battery is currently being calibrated.
Solid Orange	The inserted battery is the correct type and is currently being charged.
Solid Green	The battery is fully charged and can be removed for use.
Red blinking	The battery is too hot or too cold to be charged without damage. If the battery is too cold it will be charged as soon as it is warmed up sufficiently. If the battery is too hot it should be removed to cool down.
Solid Red	Either: - The battery is badly damaged and needs to be replaced or - it is a conventional battery which can not be charged

CLEANING

- Unplug the power supply from the mains socket.
- Wipe the battery charger and the power supply with a soft cloth dampened with a non-abrasive hospital disinfectant. Follow applicable hospital protocol for cleaning.
- Dry with a lint-free towel or medical-grade compressed air.

THI - Total Healthcare Innovation GmbH Gewerbestraße 4
 9181 Feistritz im Rosental AUSTRIA
 phone: +43 4228 30100
 fax: +43 4228 30100-20
 mail: office@thigmbh.at
 web: www.thigmbh.at

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