BATTERY PACK

INSTRUCTIONS FOR USE

REF 60401 BATTERY PACK

ENGLISH (EN) - DEUTSCH (DE) - ITALIANO (IT)

INSTRUCTIONS FOR USE - ENGLISH

IMPORTANT INFORMATION

The words **WARNING**, **CAUTION**, and **NOTE** have special meaning and should be reviewed.

WARNING	A WARNING highlights a safety-related issue. ALWAYS comply with this information to prevent patient and/or healthcare staff injury.
CAUTION	A CAUTION highlights a product reliability issue. ALWAYS comply with this information to prevent product damage.
NOTE	A NOTE supplements and/or clarifies procedural information.

COMPLIANCE STATEMENTS

This rechargeable Lithium Ion battery pack REF 60401 has been tested and has met the requirements of the following applicable regulations:

- IEC 62133:2002
- FCC Title 47 CFR, Part 15, Class B/ IECIS-003, IV edition
- ROHS Directive 2002/95/EC
- Batteries Directive 2006/66/EC and subsequent amendments
- EMC Directive 2004/108/EC
- UN Transportation Testing – UN DOT 38.3 (T1-T8) Lithium Batteries

INDICATIONS FOR USE

This battery pack is designed for use with the **OXYGEN** and **Vivi**® System. These Systems are intended to be worn by surgical personnel to provide a barrier between the operating environment and the surgical personnel in order to help protect against contamination and/or exposure of infectious body fluids and harmful microorganisms.

FOR USE WITH

- Use this battery pack to power:
 - REF 80100 **Vivi**® Helmet,
 - REF 80120 **Vivi**® Helmet w/ Headlight,
 - REF 70100 **OXYGEN** Aerolyte Helmet XS-XL,
 - REF 70110 **OXYGEN** Aerolyte Helmet M-4XL,
 - REF 70120 **OXYGEN** Aerolyte LED Helmet XS-XL,
 - REF 70130 **OXYGEN** Aerolyte LED Helmet M-4XL.
- Charge this battery pack only with REF 60500 Battery Charger.

SYMBOL DEFINITIONS

These symbols are located on the device and/or labels and are defined as follows:

SYMBOL	DEFINITION
	Caution, consult accompanying documents
	Waste Electrical and Electronic Equipment Directive (WEEE)
 Li-ion	Recyclable Product (see section 'Disposal/Recycling')
	Temperature Limitation
	Direct Current (DC)
	CE Mark
	Serial Number
	Catalog number
	Manufacturer

TECHNICAL SPECIFICATION

MODEL	REF 60401
Description	Rechargeable Battery Pack
European Conformity	CE
Cell chemistry	Lithium Ion
Voltage (nominal)	14.8 V (DC)
Capacity (rated)	2.950 mAh (43.6 Wh)
Cycle life characteristics	> 80 % of rated capacity after 400 duty cycles
Ingress Protection (IP)	IPX0
Mode of operation	Continuous
Charging time	4 hours
Battery life	> 5 hours (with REF 80100 Vivi ® Helmet) > 3 ½ hours (with REF 80120 Vivi ® Helmet w/Headlight) > 6 hours (with REF 70100, 70110 OXYGEN Aerolyte Helmets) > 4 ½ hours (with REF 70120, 70130 OXYGEN Aerolyte LED Helmets)
Mass	250 g (8.8 oz)

LIMITATIONS OF USE

WARNINGS:

- Upon delivery and before each use, inspect the battery pack to check for any eventual damage. DO NOT drain the battery if it seems damaged. DO NOT use a battery that emits odors, generates heat, or shows discoloration or leakage.
- DO NOT expose the battery pack to heat, fire, or shock.
- DO NOT heat the battery over 80 °C (176 °F).
- Exposure to high temperatures can reduce battery capacity and performance.
- DO NOT expose the battery to sources of heat or sunlight.
- DO NOT use this equipment in the presence of mixtures containing anesthetic substances, flammable air, oxygen or nitrous oxide.
- Unless otherwise indicated, DO NOT sterilize or immerse the battery pack in water, solvents, lubricants or other cleansing chemicals, or even expose them to those substances. Keep the battery clean and dry.
- DO NOT short-circuit, crush, open, tear, incinerate, or disassemble the battery pack. Never store a bulk of batteries in a box or drawer where they could short-circuit or be short-circuited by metallic objects. Damage increases the risk of chemical burns due to electrolyte spills, fire or explosion. For replacement instructions, contact your local distributor.
- In case of leakage of liquids from the battery, avoid skin contact. In case of contact, wash the affected area with plenty of water and consult a doctor. In case of ingestion of a cell or whole battery pack, consult your doctor immediately. Keep the battery pack out of reach of children.
- DO NOT use batteries other than those specifically designed for the **OXYGEN** and **Vivi**® System. The use of any other battery may cause fire, explosion, electric shock, or injury.
- Use the battery pack only for the intended applications.
- When using medical equipment, such as this battery pack, comply with the electromagnetic compatibility (CEM) procedures provided by your hospital.

RECOMMENDATIONS FOR BATTERY HANDLING

- DO NOT remove the battery from its original packaging until it is used.
- Upon delivery, your battery is in a shipping mode and will not deliver power. Before first use, put the battery into the charger to wake it up.
- After long storage periods, a full charging/discharging cycle might be necessary to regain full battery capacity.
- Dispose the battery pack as described in the Disposal / Recycling section of this manual.
- Rechargeable batteries achieve their optimum performance at room temperature, at 20 °C ± 5 °C (68 °F ± 10 °F).
- Store the original product documentation for future reference.

CHARGING INSTRUCTIONS

- For detailed instructions, consult the operator manual of the **Vivi**® Helmet and **OXYGEN** Helmet.

WARNINGS:

- ALWAYS use the REF 60500 Battery Charger to charge and store the battery pack.

CLEANING**WARNINGS:** 

- DO NOT immerse the battery pack in liquids.
- DO NOT sterilize the battery pack.

CAUTIONS:

- DO NOT use solvents, lubricants or other chemicals, unless otherwise specified.
- DO NOT allow liquids or moisture into the electrical connections.
- DO NOT allow debris to enter electrical connections.

TO CLEAN THE BATTERY PACK

1. Disconnect the battery pack from the helmet and battery charger.
2. Wipe the battery pack with a soft cloth dampened with a non-abrasive hospital disinfectant.
3. Dry the equipment with a lint-free towel or medical-grade compressed air.

STORAGE AND TRANSPORT

Recommended storage and transport conditions: - 20 °C to +35 °C (-4 °F to +95 °F), up to 75 % relative humidity.

CAUTIONS:

- To optimize battery life, battery packs are recommended to be stored in low humidity environments, free of corrosive gases, at a temperature of < 21 °C (< 70 °F). Prolonged exposure to temperatures above 45 °C (113 °F) may compromise battery life and performance.
- Carry or store the battery pack in low dust environments.
- DO NOT expose the battery pack to extreme temperature changes.

DISPOSAL/ RECYCLING**WARNINGS:** 

- ALWAYS follow the current local recommendations and/or regulations governing environmental protection and risks associated with recycling or disposing of the equipment at the end of its useful life.
- Per the European Union (EU) Waste Electronics and Electrical Equipment (WEEE) Directive 2002/96/EC, product must be collected separately.
- DO NOT dispose of as unsorted municipal waste. Contact your local distributor for disposal information.
- Contains no hazardous substances, per RoHS standard.

NOTES:

- This battery pack contains lithium. After the end of its useful life, the battery must be disposed of by qualified recycling or hazardous materials handling operators.
- DO NOT dispose the battery pack between other solid wastes. For disposal and recycling information, refer to local operators responsible for recycling or handling hazardous materials.

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Total Healthcare Innovation GmbH
THI - Total Healthcare Innovation GmbH