SPS - HEADBELT SYSTEM

INSTRUCTIONS FOR USE

REF SP100 (XS-XL)

REF SP101 (M-4XL)

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INSTRUCTIONS FOR USE - ENGLISH

IMPORTANT INFORMATION

The words WARNING, CAUTION, and NOTE have special meaning and should be reviewed.

WARNING	A WARNING highlights a safety-related issue. ALWAYS comply with this information to prevent patient and/or healthcare staff injury.
CAUTION	A CAUTION highlights a product reliability issue. ALWAYS comply with this information to prevent product damage.
NOTE	A NOTE supplements and/or clarifies procedural information.

COMPLIANCE STATEMENT

Medical Device acc. Directive 93/42/EEC

This device is subject to the conformity assessment procedure according to annex VII of Directive 93/42/EEC.

INDICATION FOR USE

The SPS Headbelt System is a component of the SPS Protection Shield System and is intended to be worn by surgical personnel to provide a barrier between the operating environment and the surgical personnel in order to help protect against contamination and/or exposure of infectious body fluids.

CONTRAINDICATIONS

None known.

LIMITATIONS OF USE

- This protection system is not intended to be used as a respiratory protection system.
- ALWAYS use the SPS Protection Shield System in combination with a surgical face mask to increase protection levels.
- The SPS Headbelt System used without the protection shield SP250 or SP260 does not provide any barrier ability as defined in the indication for use. Use the Headbelt System ALWAYS in combination with protection shield SP250 or SP260.
- The protection shield system does provide limited protection to the face from flying particles or the splash of hazardous liquids. If the system is worn over ophthalmic spectacles, impacts may be transmitted. The protection shields are not shatterproof.
- The protection shield system is not intended for use in atmospheres immediately dangerous to life or health.
- The materials contained within this equipment are not known allergens. However, if an allergic reaction occurs, contact a physician.

FOR USE WITH

The SPS – Headbelt System is a component of the SPS Protection Shield System and is intended to be used only in combination with Protection Shields REF SP250 or REF SP260.

USER/ PATIENT SAFETY

- Only trained and experienced healthcare professionals should use this equipment. Before using any system component, or any component compatible with this system, read and understand the instructions. Pay special attention to WARNING information. Become familiar with the system components prior to use.
- If any part of the device is deemed unfit for use, the whole product should be discarded.
- This device shall not be used at extremes of temperatures.
- The healthcare professional performing any procedure is responsible for determining the appropriateness of this equipment and the specific technique used for each patient. THI, as a manufacturer, does not recommend surgical procedure or technique.
- DO NOT steam sterilize this device. Steam sterilization may lead to a loss of mechanical integrity.
- Use only THI-approved components and accessories, unless otherwise specified. DO NOT modify any component or accessory.

SYMBOL DEFINITIONS

The symbols located on the equipment and/or labelling are defined in this section.

SYMBOL	DEFINITION
	Consultare le istruzioni per l'uso
	Recyclable Product
	Manufacturer
SN	Serial number
QTY	Quantity
REF	Article number

INSTRUCTIONS

- ALWAYS wear the protection shield and the headband as a total system to obtain the intended effectiveness.
- Before and during use, ALWAYS inspect the equipment for rips, scratches, cuts, or holes. DO NOT use the equipment if damage is apparent.
- ALWAYS follow current hospital protocol governing the safe removal, handling, and disposal of potential biohazard waste.

HOW TO DON THE SPS – HEADBELT SYSTEM

1. For best comfort, please make sure that all foampads are attached correctly to the headbelt.
2. Loosen the headband with the headband adjustment knob as required by turning the knob counter clockwise.
3. Place the headband on your head and adjust it for comfort.
4. Attach the protection shield to the headband. For details, see Instruction for use of the protection shield.
5. After use, remove the protection shield from the headbelt and dispose as contaminated waste.

TO CLEAN THE SPS - HEADBELT SYSTEM

1. Remove the foampads from the headbelt.
2. Wipe the headbelt with a soft cloth dampened with a nonabrasive, hospital grade disinfectant.
3. Dry the headbelt with a lint-free towel or medical-grade compressed air.

- DO NOT sterilize any component.
- DO NOT use solvents, lubricants, or other chemicals, unless otherwise specified.

STORAGE AND HANDLING

Store the equipment within the following environmental condition value(s) throughout its useful life: Temperature: above -20 °C, below +40°C, Humidity: above 10% RH, below 75% RH. To ensure the longevity, performance and safety of this equipment, use the original packaging materials when storing or transporting this equipment.

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Total Healthcare Innovation GmbH
THI - Total Healthcare Innovation GmbH Gewerbestraße 4