INSTRUCTIONS FOR USE - ENGLISH

IMPORTANT INFORMATION

The words WARNING, CAUTION, and NOTE have special meaning and should be reviewed.

WARNING	A WARNING highlights a safety-related issue. ALWAYS comply with this information to prevent patient and/or healthcare staff injury.
CAUTION	A CAUTION highlights a product reliability issue. ALWAYS comply with this information to prevent product damage.
NOTE	A NOTE supplements and/or clarifies procedural information.

COMPLIANCE STATEMENT

Medical Device acc. Directive 93/42/EEC

This device is subject to the conformity assessment procedure according to annex VII of Directive 93/42/EEC.

INDICATION FOR USE

The SP260 protection shield is a component of the SPS Protection Shield System and is intended to be worn by surgical personnel to provide a barrier between the operating environment and the surgical personnel in order to help protect against contamination and/or exposure of infectious body fluids.

CONTRAINDICATIONS

None known.

LIMITATIONS OF USE


- This protection shield is not intended to be used as a respiratory protection system.
- This protection shield used without the SPS – Headbelt System SP100 or SP101 does not provide any barrier ability as defined in the indication for use. Use the protection shield ALWAYS in combination with SPS – Headbelt System SP100 or SP101.
- This protection shield does provide limited protection to the face from flying particles or the splash of hazardous liquids. If these lenses are worn over ophthalmic spectacles, impacts may be transmitted. The lenses are not shatterproof.
- The protection shield is not intended for use in atmospheres immediately dangerous to life or health.
- The materials contained within this device are not known allergens. However, if an allergic reaction occurs, contact a physician.

FOR USE WITH

The SP260 Protection Shield is a component of the SPS Protection Shield System and is intended to be used in combination with the SPS Headbelt Systems REF SP100 or REF SP101.

USER/ PATIENT SAFETY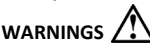

- Only trained and experienced healthcare professionals should use this equipment. Before using any system component, or any component compatible with this system, read and understand the instructions. Become familiar with the system components prior to use.
- If any part of the device is deemed unfit for use, the whole product should be discarded.
- This device shall not be used at extremes of temperatures.
- The healthcare professional performing any procedure is responsible for determining the appropriateness of this equipment and the specific technique used for each patient. THI, as a manufacturer, does not recommend surgical procedure or technique.
- DO NOT reprocess or repack this device. This device is intended for a single use only.
- DO NOT clean or sterilize this device. Cleaning or sterilization may significantly affect the optical quality or loss of mechanical integrity. Use only THI-approved components and accessories, unless otherwise specified. DO NOT modify any component or accessory.

SYMBOL DEFINITIONS

The symbols located on the equipment and/or labelling are defined in this section.

SYMBOL	DEFINITION
	Consult instructions for use
	Do not use if package is damaged
	Manufacturer
LOT	Batch code
QTY	Quantity
REF	Article number
	Non sterile
	Keep away from direct sunlight

INSTRUCTIONS

- ALWAYS wear the protection shield and the headband as a total system to obtain the intended effectiveness.
- Before and during use, ALWAYS inspect the equipment for rips, scratches, cuts, or holes. DO NOT use the equipment if damage is apparent.
- ALWAYS follow current hospital protocol governing the safe removal, handling, and disposal of potential biohazard waste.

HOW TO INSTALL THE PROTECTION SHIELD

- Open the packaging and remove the protection shield from the packaging.
- Point the hook-and-loop pieces on the protection shield upwards and towards you.
- Carefully connect the hook-and-loop pieces onto their corresponding pieces on the headbelt (REF SP100 and SP101). Ensure that you have created a tight fit between the protection shield and the headbelt.

STORAGE AND HANDLING

Store the equipment within the specified environmental condition value(s) throughout its useful life. See the product label for details. To ensure the longevity, performance and safety of this equipment, use the original packaging materials when storing or transporting this equipment.

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Total Healthcare Innovation GmbH
THI - Total Healthcare Innovation GmbH Gewerbestraße 4